


# NXTEK™ ELIMINATE MALARIA PF

10 times more sensitive than current malaria rapid diagnostic tests for the detection of *Plasmodium falciparum* HRP-II antigen.<sup>1</sup>

Enable significantly better identification of asymptomatic individuals with low density parasitemia, with and without evident symptoms of malaria infection.

## CURRENT MALARIA RDT

Cannot detect infection in asymptomatic individuals with low parasitemia

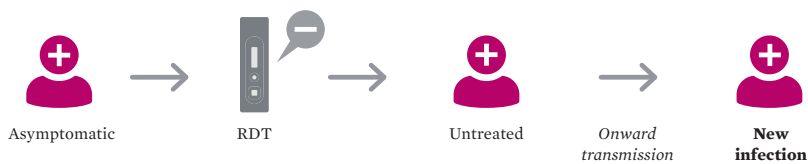

## NXTEK™ ELIMINATE MALARIA PF

Can detect infection in asymptomatic individuals with low parasitemia




The systematic identification and treatment of infected individuals will reduce the malaria reservoir in the community and thereby drive down transmission.




# NxTek™ Eliminate Malaria Pf

IMPROVED IDENTIFICATION OF INDIVIDUALS WITH LOW PARASITEMIA IN ASYMPTOMATIC POPULATIONS\*

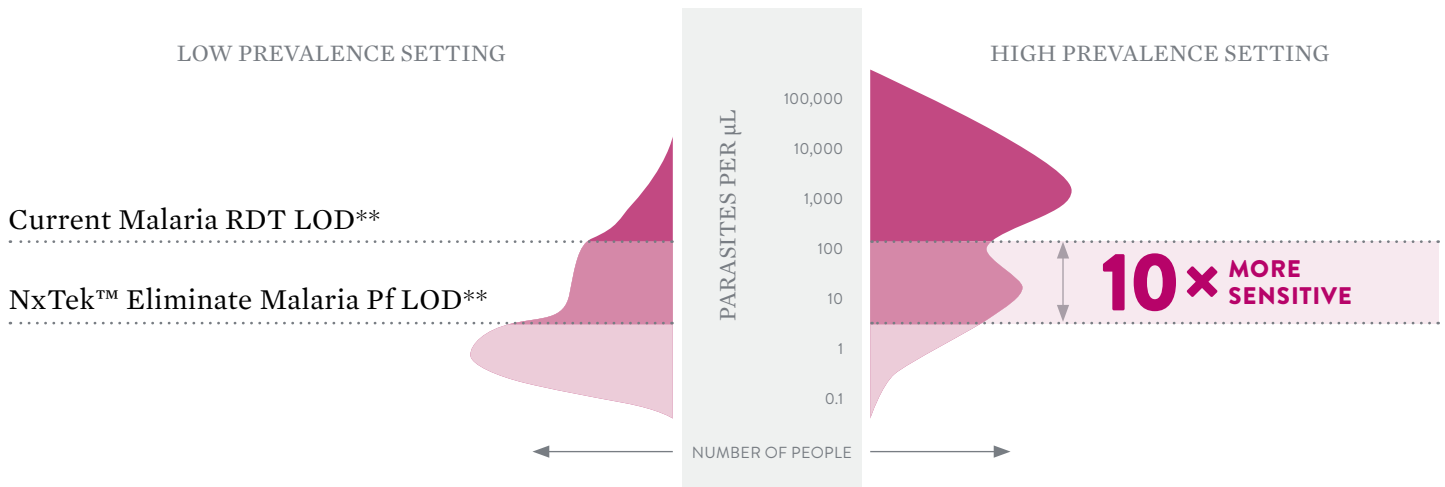


## EASY TO USE


**1 Add specimen.**  
Dispense 5 µL of whole blood into the round specimen well.

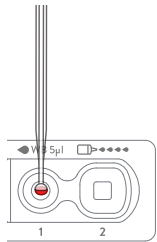

**2 Add assay diluent.**  
Dispense 4 drops of assay diluent into the square assay diluent well.

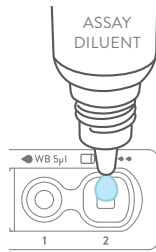

**3 Read the test result in 20 minutes.**

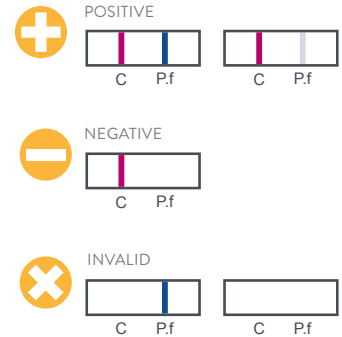

## ORDER INFORMATION\*\*\*

PRODUCT	CATALOGUE NO.	DESCRIPTION
NXTEK™ ELIMINATE MALARIA PF	05FK140	STANDARD TEST PACK (25 TESTS/KIT) <sup>†</sup> LANCET OPTION: STERILE LANCET
	05FK141	STANDARD TEST PACK (25 TESTS/KIT) <sup>†</sup> LANCET OPTION: SAFETY LANCET
	05FK142	POCT TYPE BUNDLE PACK (1T/KIT × 25 EACH) <sup>††</sup> – LANCET OPTION: SAFETY LANCET
	05FK143	POCT TYPE BUNDLE PACK (1T/KIT × 25 EACH) <sup>††</sup> – LANCET OPTION: STERILE LANCET

FOR MORE INFORMATION, PLEASE CONTACT YOUR LOCAL REPRESENTATIVE.

\*Adapted with permission from PATH.<sup>1</sup>

\*\*Limit of detection.

\*\*\*Not cleared/approved or for sale in the USA.

<sup>†</sup>Standard test pack (25 tests/kit): 25 test devices, 1 assay diluent, 25 disposable inverted cups, 25 alcohol swabs, 1 pack insert and 25 lancets.

<sup>††</sup>Point-of-care testing (POCT) type (1 test/kit): 1 test device, 1 assay diluent, 1 disposable inverted cup, 1 alcohol swab and 1 lancet. A bundle pack includes 25 POCTs and 1 pack insert.

1. Das, S. et al. Performance of a High-Sensitivity Rapid Diagnostic Test for Plasmodium falciparum Malaria in Asymptomatic Individuals from Uganda and Myanmar and Naive Human Challenge Infections. *Am J Trop Med Hyg.* 2017. doi:10.4269/ajtmh.17-0245.

© 2020 Abbott. All Rights Reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. 120006028-01 2/20

